MDS GUARD+

Cold High-Performance Disinfectant For Surgical & Thermo-Sensitive Instruments

Product Information:

MDS GUARD+ is a ready to use cold high-performance disinfectant for disinfecting thermo-sensitive devices as well as non/semi-critical medical and surgical instruments when needed.

MDS GUARD+ is a blend of fifth generation quaternary ammonium compounds with strong oxidizing agent and effective catalysts, providing a bactericidal, virucidal and fungicidal action including the elimination of non-enveloped viruses and mycobacterium tuberculosis.

- Excellent material compatibility; non-caustic, non-staining, non-toxic.
- Effective and fast acting.
- Aldehyde, chlorine and phenol free.
- Economical; lasts 14 days of moderate use and 7 days for heavy use.
- Contains corrosion inhibitors.

Recommended Uses:

- Cold Advanced disinfectant for thermo-sensitive instruments such as endoscopes and instruments with rubber, plastic, glass and silicone.
- Cold advanced disinfectant for non/semi-critical medical and surgical instruments in the absence of heat exposure possibility.
- Cold advanced disinfectant for rust-sensitive instruments.
- Disinfection of impression molds and dental prosthetic devices.
- A temporary post-disinfection immersion medium for semi-critical medical instruments prior to immediate use.

Instructions For Use:

- Completely submerge pre-cleaned and rinsed (non/semi-critical instruments) in a disinfection tray containing MDS GUARD+. Make sure all surfaces are in contact with the disinfectant solution.
- Cover the tray.
- Contact time: 5 minutes for disinfecting non-critical instruments (scissors, forceps, examination devices..) / 10 minutes for disinfecting semi-critical instruments (endoscopes, respiratory and anesthesia equipment, dental instruments..).
- Remove instruments using post sterilization procedures. No rinsing required.
- Replace the solution after 14 days of moderate use or 7 days of heavy use (the disinfection container must be tightly closed after the disinfection process is completed and not exposed to heat or light).

Anti-Microbial Efficacy :

ACTIVITY SPECTRUM	STANDARD	STRAINS	CONTACT TIME
Bactericidal (Gram +/-)	EN 1276	Staphylococcus aureus Enterococcus faecalis Pseudomonas aeruginosa Escherichia coli Salmonella typhimurium	5 min
	EN 13727	Staphylococcus aureus Enterococcus hirae Pseudomonas aeruginosa Escherichia coli	
Virucidal (Enveloped/ Non-enveloped)	EN 14476	Vaccinia, HIV, HBV, HTLV, HDV, Influenza virus, Coronavirus (e.g. SARS, MERS), Herpesviridae, Hepatitis C, Adenovirus, Poliovirus, Murine norovirus	10 min
Fungicidal / Yeasticidal	EN 13624	Candida albicans	10 min
Tuberculocidal / Mycobactericidal	EN 14348	Mycobacterium avium Mycobacterium terrae	15 min

WARNING:

MDS GUARD+ is non-toxic to higher organisms; however, it remains a proven chemical disinfectant and should not be used by uninformed persons. Avoid eye contact. Do not ingest. Wear protective gloves for sensitive skin only. Incompatible with anionic detergents and natural soaps.

FIRST AID:

In case of skin contact, flush area with water.
In case of eye contact, flush abundantly with water.
Redness if any should dissipate within 30 minutes.
If irritation persists, consult a physician.
In case of ingest, induce vomiting, drink a large glass of milk and seek medical advice.

STORAGE:

Temperature +5 ° C to + 30 ° C. Store out of reach of direct sunlight.
Keep out of reach of children.

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Composition:

0.2% Alkyl Dimethyl Benzyl Ammonium Chloride, 0.2% Didecyl dimethylammonium Chloride, 3% Hydrogen Peroxide, Stabilizer, Catalysts, Anticorrosive substances, Deionized water.

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