PuroOxi

MDS GUARD+

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1L 5L

Product Information:

- * MDS Guard+ is a ready to use cold sterilant for disinfecting and sterilizing thermo-sensitive devices as well as medical and surgical instruments when needed.
- * MDS Guard+ is also used as a holding medium to maintain instrument sterility.

MDS Guard +

- * Log 6 reduction of Bacillus Subtilis Spores in 10 minutes.
- * Aldehyde ; chlorine and phenol free.
- * Non-toxic and non-irritant.
- * Excellent material compatibility; non-caustic, non-staining and non corrosive.
- * Contains corrosion inhibitors.
- * Economical ; lasts 28 days for moderate use & 21 days for heavy use.

Recommended Uses:

- * Cold sterilization of thermo-sensitive instruments such as instruments with rubber , plastic , glass & silicone.
- * Cold sterilization of medical and surgical instruments in the absence of heat sterilization.
- * Cold sterilization and disinfection of rust-sensitive instruments.
- * Disinfection of impression molds and dental prosthetic devices.
- * Holding medium to maintain instrument sterility.

Instructions For Use:

- * Completely submerge pre-cleaned and rinsed instruments in a sterilization tray containing MDS GUARD+.
- Make sure the instruments are completely submerged all surfaces in contact with MDS GUARD+.
- * Cover the tray.
- * Contact time : 10 minutes for sterilization/5 minutes for disinfection.
- * Remove instruments using proper post sterilization procedures, No rinsing required.
- * Replace the solution after 28 days of moderate use and 21 days of heavy use.

Composition:

ADBAC blend (0.2%), DDAC (0.2%), rust inhibitor (0.2%), Catalysts (0.2%), inert ingredients (99.2%).

Anti-Microbial Efficacy*:

Bactericidal: Enterobacter aerogenes, Brevibacterium, Enterococcus faecalis, Klebsiella pneumonia, Shigella dysenteriae, Salmonella schottmuelleri, Staphylococcus aureus, Pseudomonas aeruginosa, Staphylococcus aureus (MRSA), Escherichia coli, Salmonella enterica, Staphylococcus aureus (VISA), Streptococcus pyogenes, Coliforms, Chlamydia trachomatis, Actinomyces naeslundii, Listeria monocytogenes, Mycobacterium tuberculosis.

Fungicidal: Aspergillus niger & flavus, Trichophyton mentagrophytes, Candida albicans.

Virucidal: Influenza A2/Japan, Herpes Simplex type 1, Adenovirus type 2, Vaccinia virus, HIV-1 protease, HBV, HCV, Human Coronavirus, Norovirus, SARS, BVDV, Feline calicivirus.

Microbicidal: Dermatophagoides pteronyssinus.

Sporocidal: Bacillus subtilis & cereus.

- * Tested on the active ingredient blend contained in this product.

WARNING :

Avoid eye contact. Do not ingest
Incompatible with anionic detergents and natural soaps.

*First Aid

In case of eye contact , flush abundantly with water. Redness if any should dissipate within 30 min. If irritation persists consult a physician.

In case of ingestion , induce vomiting , drink a large glass of milk and seek medical advice.

*Storage

Temperature : +5°C to +40°C

Store out of reach of direct sunlight. Keep out reach of children.

Lebanon Distributor
ProMedical Supplies Est
Beirut, Cornish Al-Mazraa
Phone : +961 1307279
Fax : +961 1305279
E-mail : Promedical.supplies14@gmail.com

MDS GUARD+

Cold Sterilant for Surgical
& Thermo-Sensitive Instruments

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