OPTI-SPRAY

Quick Spraying Surface Sanitizer & Disinfectant

Product Information:

OPTI-SPRAY is a ready to use disinfectant for hard surfaces, surfaces of equipment and devices as well as soft surfaces and can be used through wiping or spraying. **OPTI-SPRAY** is blend of fifth generation quaternary ammonium compounds with strong oxidizing agent, ethyl alcohol and effective catalysts, providing a wide anti-microbial activity against bacteria, viruses and fungi including the elimination of non-enveloped viruses and mycobacterium tuberculosis, as well as residual action allowing continuous disinfection after use.

- Safe, aldehyde, chlorine and phenol free.
- Non-caustic, non-staining and non-corrosive.
- Effective and fast acting.
- Good material compatibility with metals, painted surfaces, synthetic surfaces (vinyl, fiber, PVC), rubber and glass.
- Slight lemon fragrance.

Recommended Uses:

Disinfection of all hard and soft surfaces, particularly high touch surfaces in critical and non-critical areas in healthcare facilities, in the clinic such as countertops, workstations, dental units, sinks, spittoons, cabinets, floors, walls, toilets, etc.

- Disinfection of non/semi-critical medical, dental, and surgical instruments, equipment and devices, materials and fabrics.
- Disinfection of impression molds and dental prosthetics.
- Air purification.

Instructions For Use:

- **For surfaces:**
 - * Spray directly and evenly on pre-cleaned and rinsed surfaces from a distance of 30-50 cm, or evenly on a clean preferable single use cloth and wipe the surfaces to be disinfected uniformly covering all areas.
 - * Application rate: 20 – 40 ml / m².
 - * Rub with a mop for large surfaces.
 - * Allow to air dry or ensure 5 minutes contact time to disinfect then wipe dry with a sterile cloth or a clean cloth sprayed with OPTI-SPRAY.
 - * Do not rinse.
- **For Furniture:** * Spray it on the surfaces to disinfect and allow it to dry.
- **For Air:** * Spray into the air.

Anti-Microbial Efficacy:

ACTIVITY SPECTRUM	STANDARD	STRAINS	CONTACT TIME
Bactericidal (Gram +/-)	EN 1276	Staphylococcus aureus Enterococcus faecalis Pseudomonas aeruginosa Escherichia coli Salmonella typhimurium	1 min
	EN 13727	Staphylococcus aureus Enterococcus hirae Pseudomonas aeruginosa Escherichia coli	
Virucidal (Enveloped/ Non-enveloped)	EN 14476	Vaccinia, HIV, HBV, HTLV, HDV, Influenza virus, Coronavirus (e.g. SARS, MERS), Herpesviridae, Hepatitis C, Adenovirus, Poliovirus, Murine norovirus	3 min
Fungicidal / Yeasticidal	EN 13624	Candida albicans	3 min
	EN 1275	Aspergillus brasiliensis	
Tuberculocidal / Mycobactericidal	EN 14348	Mycobacterium avium Mycobacterium terrae	5 min

WARNING:

Avoid eye contact. Do not ingest.
Incompatible with anionic detergents, chlorine, and natural soap.

FIRST AID:

In case of skin contact, rinse the area thoroughly with water.
In case of ingestion, do not induce vomiting, drink a glass of water and seek medical advice.
In case of eye contact, flush abundantly with water, redness if any should dissipate within 30 minutes, if irritation persists, consult a physician.

STORAGE:

Temperature +5°C to +30°C. Store out of reach of direct sunlight.
Keep out of reach of children.

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