PuroOxi

OPTI-SPRAY

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*Air
*Surfaces

Wide Spectrum and Impact
Versatile Applications
Lemon Fragrance

650
ml

5L

OPTI-SPRAY

Quick Spraying
Surface Sanitizer & Disinfectant
BACTERICIDAL - PESTICIDE - VIRUCIDE
FUNGICIDE - DEODORIZER
Hospital / Medical / Surgical / Veterinary

Product Information:

- * OPTI-SPRAY is a ready to use disinfectant for hard surfaces , surfaces of equipment and devices as well as soft surfaces.
- * OPTI-SPRAY can be used through wiping or spraying.

OPTI-SPRAY

- * Safe ; aldehyde , chlorine and phenol free.
- * Non-toxic and non-irritant.
- * Good material compatibility with metals , painted surfaces, synthetic surfaces (vinyl , fiber , PVC) , rubber and glass.
- * Non-caustic , non-staining and non-corrosive.
- * Fast and effective.
- * Slight lemon fragrance.

Recommended uses:

- * Disinfection of all hard and soft surfaces in the clinic such as dental units, spittoons , sinks , cabinets , etc.
- * Surface disinfection of equipment and devices.
- * Disinfection of impression molds and dental prosthetics.
- * Air Sanitization.

Instructions For Use:

- * Spray directly and evenly on pre-cleaned and rinsed surfaces from a distance of 30-50cm , or evenly on a clean preferably single use cloth and wipe the surfaces to be disinfected uniformly covering all areas. application rate :20-40ml/m² . rub with a mop for large surfaces.
- * Allow to air dry or ensure 5 minutes contact time to disinfect then wipe dry with a sterile cloth or a clean cloth sprayed with OPTI-SPRAY.
- * Do not rinse.

Composition:

ADBAC blend (0.08%), DDAC (0.02%), rust inhibitor (0.03%), Catalysts (0.07%),perfume (0.08%), inert ingredients (99.72%).

Bactericidal: Pseudomonas aeruginosa , Staphylococcus aureus , Chlamydia trachomatis ,Enterobacter cloacae , Actinomyces naeslundii , Ureaplasma urealyticum , Streptococcus faecalis & mutans , Lactobacillus leichmanii & paracasei , Enterococcus hirae , Escherichia coli , Coliforms, MRSA , Salmonella typhi & choleraesuis , Klebsiella pneumoniae , Listeria monocytogenes , Bacillus subtilis & cereus and Mycobacterium tuberculosis.

Fungicidal including fungal spores: Aspergillus niger & flavus , Candida albicans and Trichophyton mentagrophytes.

Virucidal: HIV , Polio , Herpes simplex , Influenza , Cytomegalovirus , Hepatitis A&B , rotavirus , Foot & Mouth Disease , Avian Influenza , Norovirus and SAR.

Microbiocidal: Dermatophagides pteronyssinus.

WARNING :

Avoid eye contact. Do not ingest.

Incompatible with anionic detergents and natural soaps.

*First Aid

In case of eye contact , flush eye with water. Redness if any should dissipate within 30 min. If irritation persists consult a physician.

In case of ingestion , do not induce vomiting , drink a glass of water and seek medical advice.

*Storage

Temperature : +5°C to +40°C

Store out of reach of direct sunlight. Keep out reach of children.

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