PURIVAC

Concentrated Cleaner & High-Level Disinfectant For Suction, Spittoons and Hemodialysis Apparatus

Product Information:

PURIVAC is a concentrated non-foaming solution for cleaning, disinfecting and deodorizing suction systems, separators, and spittoons in dental clinics, Hemodialysis apparatus from germs, coagulated blood, organic residue and soiling limescale of biofilm.

- Provides a wide anti-microbial activity against bacteria, viruses (including non-enveloped), fungi, bacterial spores and parasites.
- Aldehyde and phenol free.
- Odor control in suction lines. Prevents blockage.
- Economical to use.

Recommended Uses:

- Suction lines, spittoons, separators and water units.
- Hemodialysis apparatus.

Instructions For Use:

- Shake well before use.
- Dilute PURIVAC at 2:100 by adding 20ml to 1L of water (2 bottle's cap).
- Pour or aspirate 1L of the solution into the system or use the amount indicated in the maintenance instructions of the equipment.
- Soak for 10 minutes; 15 minutes for heavily soiled equipment.
- Flush out.
- Rinse the system once with fresh water.
- * Aspirate after each use or at least twice a day (mid-day & at the end of the working day).
- * For use in hemodialysis apparatus, follow the manufacturer's instructions.
- * Stability of the unused diluted solution is 7 days.

Anti-Microbial Efficacy :

ACTIVITY SPECTRUM	STANDARD	STRAINS	CONTACT TIME
Bactericidal (Gram +/-)	EN 13727	Staphylococcus aureus Enterococcus hirae Pseudomonas aeruginosa Escherichia coli	5 min
Virucidal (Enveloped/ Non-enveloped)	EN 14476	Vaccinia, HIV, HBV, HTLV, HDV, Influenza virus, Coronavirus (e.g. SARS, MERS), Herpesviridae, Hepatitis C, Adenovirus, Poliovirus, Murine norovirus	10 min
Fungicidal / Yeasticidal	EN 13624	Candida albicans	10 min
	EN 1275	Aspergillus brasiliensis	
Bacterial Spores	EN 13704	Bacillus Subtilis	15 min
Tuberculocidal / Mycobactericidal	EN 14348	Mycobacterium terrae	10 min

WARNING:

PURIVAC is a concentrated product with highly alkaline pH. Do not mix with acids. Safety measures should be taken to avoid inhalation or skin exposure, protective rubber or neoprene gloves are recommended when using the concentrate. Do not ingest.

FIRST AID:

In case of skin contact, rinse thoroughly with water.
In case of eye contact, rinse immediately with plenty of water for several minutes, and seek medical attention.
In case of ingestion, rinse mouth, do not induce vomiting and seek medical advice.
In case of inhalation, remove person to fresh air and seek medical attention immediately.

STORAGE:

Store in original container locked up in between +5 °C to +30 °C away from heat and direct sunlight. Keep out of reach of children.

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