SONIDET CLEANER

Concentrated Enzymatic Cleaner and Disinfectant For Medical & Surgical Instruments

Product Information:

- SONIDET CLEANER is a concentrated enzymatic cleaner and disinfectant that removes organic and microbial loading from instruments prior to sterilization.
- SONIDET CLEANER is a blend of fifth-generation quaternary ammonium compounds with ethyl alcohol, effective catalysts, 4 enzymes (amylase, protease, lipase, cellulase) and balanced system of detergents and alkali that provide a wide anti-microbial activity against bacteria, viruses and fungi, as well as a residual action that allows continuous disinfection after use and effectively removes organic loading.
- Effectively dissolves organic soiling like blood, oil, protein and needs no scrubbing.
- Cleans and disinfects in 10 minutes.
- Compatible with metals, glass and rubber.
- Contains corrosion inhibitors to protect instruments from rust.
- Aldehyde, chlorine and phenol free.
- Non-toxic and non-irritant.

Recommended Uses:

- Pre-cleaning of medical and surgical instruments, equipment and endoscopes. Thanks to its four enzymes content those provide more effective cleaning power.
- Can be used as ultrasonic bath solution.
- Any rust sensitive medical or surgical instrument or surfaces.
- Cleaning and disinfecting of impression molds and dental prosthetics, as well as non/semi-critical medical and surgical instruments.

Instructions For Use:

- Rinse instruments with water.
- Shake well before use, prepare the solution by adding SONIDET CLEANER at 1:25 means that 40 ml SONIDET CLEANER to 1L of water.
- Completely soak instruments in the bath containing the prepared diluted solution.
- Contact time:
 - * 20 minutes for heavily soiled instruments (Stirring the solution may be necessary).
 - * 10 minutes for moderately soiled instruments.
- Remove the instruments, rinse well and dry.
- Discard the used solution.
- * Stability of the unused diluted solution is 6 days.

Anti-Microbial Efficacy :

ACTIVITY SPECTRUM	STANDARD	STRAINS	CONTACT TIME
Bactericidal (Gram +/-)	EN 1276	Staphylococcus aureus Enterococcus faecalis Pseudomonas aeruginosa Escherichia coli Salmonella typhimurium	5 min
	EN 13727	Staphylococcus aureus Enterococcus hirae Pseudomonas aeruginosa Escherichia coli	
Virucidal (Unenveloped/ Non-enveloped)	EN 14476	Vaccinia, HIV, HBV, HTLV, HDV, Influenza virus, Coronavirus (e.g. SARS, MERS), Herpesviridae, Hepatitis C, Adenovirus, Poliovirus, Murine norovirus	10 min
Fungicidal / Yeastcidal	EN 13624	Candida albicans	10 min

WARNING:

SONIDET CLEANER is a concentrated product with alkaline pH; uninformed persons should not use it. Avoid eye contact. Do not ingest. Incompatible with anionic detergents and natural soap. Protective rubber or neoprene gloves are recommended for sensitive skin, only when using the concentrate.

FIRST AID:

In case of skin contact with the concentrate, thoroughly rinse the area with water. In case of eye contact, flush eye with water. Redness if any should dissipate within 30 min. If irritation persists, consult a physician. In case of ingestion, do not induce vomiting, wash out mouth, drink a large glass of milk mixed with a raw egg and seek medical advice.

STORAGE:

Temperature: +5 °C to +40 °C. Store out of reach of direct sunlight. Keep out of reach of children.

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Composition:

1.4% Alkyl Dimethyl Benzyl Ammonium Chloride, 0.6% Didecyl dimethyl ammonium Chloride, 10% Ethyl Alcohol, 5% No-ionic Surfactants, 2.5% Anticoagulant, Enzymes, Catalysts, Anticorrosive substances, Deionized water.

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