SURF GUARD

Concentrated Surface Cleaner And Disinfectant

Product Information:

SURF GUARD is a concentrated solution for the simultaneous cleaning and disinfecting of hard-washable surfaces.

SURF GUARD is a blend of fifth-generation quaternary ammonium compounds with isopropyl alcohol, effective catalysts and balanced system of detergents and alkali that provide a wide anti-microbial activity against bacteria, viruses and fungi, as well as a residual action that allows continuous disinfection after use and effectively removes organic loading.

- Dual action, deep cleaning and effective disinfection.
- Disinfects in 10 minutes.
- Non-toxic and non-irritant.
- Safe; aldehyde, chlorine and phenol free.
- Good material compatibility with synthetic surfaces (PVC, vinyl), painted surfaces and non-corrosive metallic surfaces.

Recommended Uses:

- Safe and effective cleaning and decontamination of surfaces such as tiles, synthetic surfaces (PVC, VINYLE and fiber), painted surfaces and non-corrosive metallic surfaces, including food contact surfaces.
- Cleaning and disinfection of all non-critical areas and surfaces in critical areas in healthcare facilities (operating rooms, intensive care departments, transplant and dialysis departments, emergency department, patient rooms, clinics, corridors, and waiting rooms).
- Safe and effective cleaning of blood spills and other contaminated body fluids on surfaces and medical devices.
- Eradication of molds, mildew and fungal growth in buildings and bathrooms.

Instructions For Use:

- Shake well before use.
- Dilution: 1:100 up to 1:250 or by adding 4 – 10ml to 1L of water (one bottle's cap = 10ml).
- Apply **SURF GUARD** solution evenly on surfaces and rub uniformly with a mop or a high-pressure washer.
- Allow to dry or allow 10 minutes contact time to disinfect before rinsing or wiping with pure water when protocols suggest.
- * Stability of unused diluted solution is 6 days.
- * Always disinfect surfaces after wiping with pure water particularly in hygiene sensitive and critical areas.

Anti-Microbial Efficacy:

ACTIVITY SPECTRUM	STANDARD	STRAINS	CONTACT TIME
Bactericidal (Gram +/-)	EN 1276	Staphylococcus aureus Enterococcus faecalis Pseudomonas aeruginosa Escherichia coli Salmonella typhimurium	5 min
	EN 13727	Staphylococcus aureus Enterococcus hirae Pseudomonas aeruginosa Escherichia coli	
Virucidal (Enveloped/ Non-enveloped)	EN 14476	Vaccinia, HIV, HBV, HTLV, HDV, Influenza virus, Coronavirus (e.g. SARS, MERS), Herpesviridae, Hepatitis C, Adenovirus, Poliovirus, Murine norovirus	10 min
Fungicidal / Yeasticidal	EN 13624	Candida albicans	10 min
	EN 1275	Aspergillus brasiliensis	

WARNING:

SURF GUARD is a concentrated product with alkaline pH; uninformed persons should not use it. Avoid eye contact. Do not ingest. Protective rubber or neoprene gloves are recommended for sensitive skin, only when using the concentrate.

Incompatible with anionic detergents, chlorine and natural soaps.

First Aid:

In case of skin contact with the concentrate, rinse the area thoroughly with water.

In case of ingestion, do not induce vomiting, drink a glass of milk mixed with a raw egg and seek medical advice.

In case of eye contact, flush with water, redness if any should dissipate with 30 minutes, if irritation persists, consult a physician.

STORAGE:

Temperature +5°C to +40°C. Store out of reach of direct sunlight.

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